



Diversey has been, and always will be, a pioneer and facilitator for life. We constantly deliver revolutionary cleaning and hygiene technologies that provide total confidence to our customers across all of our global sectors.

Diversey is headquartered in Fort Mill, SC, USA. For more information, visit www.diversey.com or follow us on social media.



Diversey

CLEANPatch

Medical surface repair patch
for hospital beds and stretchers



The Problem...

Damaged mattresses are common in healthcare facilities, and have been shown to harbor pathogens that contribute to Healthcare Associated Infections.

Studies in Canada, the US and UK reveal that 26% to 48% of mattresses in circulation are damaged.*

www.diversey.com

© 2020 Diversey, Inc. All Rights Reserved. 39481-LIT-en (20/010)

CLEANPatch is a registered trademark of Surface Medical, Inc., used under license.

Previous Alternatives

Replace Mattress	<ul style="list-style-type: none"> ✓ Validated ✗ Unaffordable ✗ Inconvenient
Repair with Tape	<ul style="list-style-type: none"> ✓ Inexpensive ✗ Unproven ✗ Not Durable
Do Nothing	<ul style="list-style-type: none"> ✗ Risk Patient Safety



Why CleanPatch®?

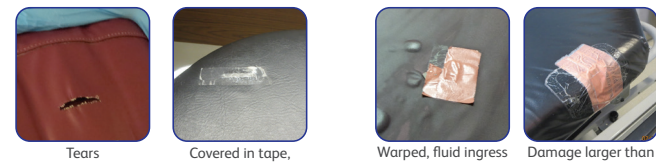
CleanPatch® is the first medical surface repair patch for hospital beds and stretchers. It restores damaged mattresses to an intact and hygienic state.

- Clinically Tested**
 - Validated by independent Infection Prevention Professionals
 - Durable and impervious to fluids under rigorous conditions
 - Fully cleanable with hospital disinfectants
- Safe and Simple**
 - Early intervention reduces the risk of cross contamination from damaged mattresses
 - Made of latex-free, biocompatible materials
 - Simple peel and stick application, adheres in seconds
- Cost Effective**
 - Extends the life of mattresses and stretchers
 - Proactively improves patient safety
 - Significant savings from equipment repair versus replacement

Getting Started:

- Determine who will manage CleanPatch® applications in each clinical area.
- Inspect mattresses during, or after, every terminal cleaning.
- Ensure CleanPatch® is readily available at the point of use.
- Review the CleanPatch® application instructions and Implementation Guide.
- Keep track of repairs to monitor the durability and the savings.

Use CleanPatch® as part of a Mattress Inspection and Repair program

	✓ Use CleanPatch®	✗ Do Not Use CleanPatch®
Mattress Age and Condition	Usable within expected lifespan	No longer suitable for use
Damage Size and Location	Less than 4 inches (10 cm) Flat areas or over edges (2-D)	More than 4 inches (10 cm) Sharp corners (3-D)
Fluid Penetration	No signs of fluid ingress	Mattress cover is visibly stained or warped
Size of Damage	Use 1 patch per damage area	Layering multiple patches not recommended
Case Examples		

Product Information

Product #	Shape	Shape	Damage Size*
#11003	Round	2 inch (5 cm)	Up to 1/2 inch (1.3 cm)
#10003	Square	3.5 x 3.5 inch (9 x 9 cm)	Up to 2 inches (5 cm)
#12003	Rectangle	3.5 x 6 inch (9 x 15 cm)	Up to 4 inches (10 cm)
#18003	Mixed Box	2 inch (5 ea.) 3.5 inch (3 ea.) 3.5 X 6 inch (2 ea.)	See references above



To ensure product integrity,
Do Not cut CleanPatch®